

Learn how Oncotype DX AR-V7 Nucleus Detect® test may help you

The AR-V7 Nucleus Detect® test is a blood test for patients with metastatic prostate cancer. Metastatic means the cancer has spread from your prostate to other parts of your body.

Your doctor may order this test for you if your cancer is progressing and you are receiving or have received therapies like Zytiga® (abiraterone), Xtandi® (enzalutamide), or Erleada™ (apalutamide).

These therapies help slow the growth of prostate cancer—but they can stop working over time.

What is AR-V7?

Androgen hormones, such as testosterone, activate or “switch on” androgen receptors (AR) in your body, causing tumors to grow. AR-targeted therapies help slow tumor growth by reducing androgens or blocking the receptors.

Sometimes, your cancer may try to outsmart your therapy by changing or altering your receptors—AR-V7 is an example of this. If AR-V7 is found within your tumor cells, AR-targeted therapies are unlikely to work for you.

This is called having an AR-V7 positive status.

The AR-V7 Nucleus Detect test tells you and your doctor what your AR-V7 status is, which can help you decide what is next in your treatment plan.

Once the Oncotype DX AR-V7 Nucleus Detect® test has been ordered for you:

The Genomic Access Program can help you gain access to the test. If you have questions regarding your insurance coverage or would like information on this program:

- Visit OncotypeQ.com
- Call **(866) ONCOTYPE (866-662-6897)** or **877-444-9876** for assistance in Spanish

For more information about genomic testing and the AR-V7 Nucleus Detect® test, visit OncotypeDXARV7.com/patient.

Oncotype DX AR-V7 Nucleus Detect test is a collaboration between Exact Sciences and Epic Sciences™.

Exact Sciences is the world’s leading provider of cancer screening and genomic-based diagnostic tests, guiding treatment decisions for more than a million cancer patients worldwide with the Oncotype DX® cancer tests.

Epic Sciences Inc. develops novel diagnostics to improve patient lives by enabling precision medicine with noninvasive tests that profile rare cells in cancer.

Oncotype DX AR-V7 Nucleus Detect, Oncotype DX, AR-V7 Nucleus Detect, and Oncotype IQ are trademarks of Genomic Health Inc., an Exact Sciences corporation. Exact Sciences is a registered trademark of Exact Sciences corporation. All other trademarks are the properties of their respective owners.

©2020 Genomic Health Inc., an Exact Sciences corporation. All rights reserved. GHI60013_0320

oncotype DX
AR-V7 Nucleus Detect
powered by Epic Sciences

**EXACT
SCIENCES**

oncotype DX
AR-V7 Nucleus Detect
powered by Epic Sciences

**EXACT
SCIENCES**

FOR PATIENTS WITH
METASTATIC PROSTATE CANCER

**Oncotype DX AR-V7
Nucleus Detect® test:
Know when to
consider another
treatment direction**

**Available
and
covered by
Medicare**




Why this test was ordered for you

Know for sure what your options are

With the Oncotype DX AR-V7 Nucleus Detect® test, you and your doctor can easily determine your AR-V7 status. Based on your results, you and your doctor can then confidently move forward with the treatment that is right for you.

Metastatic castration-resistant prostate cancer (mCRPC)

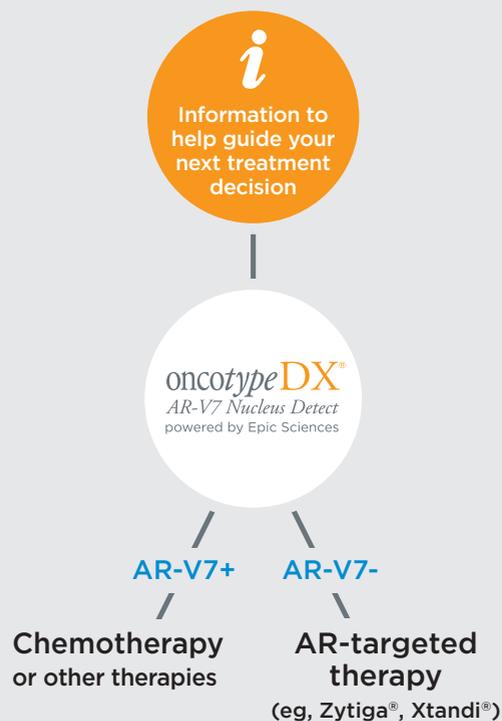

The image above shows a possible mCRPC treatment path

What your AR-V7 status means

Your AR-V7 status will tell you if AR-targeted therapies will not work for you



If you are AR-V7+

- AR-V7 was found in the nucleus of your tumor cells
- AR-targeted therapies are unlikely to work for you
- Your doctor may recommend chemotherapy or other therapies



If you are AR-V7-

- AR-V7 was **not** found in the nucleus of your tumor cells
- AR-targeted therapies may still work for you, but they may not
- Ask your doctor about your treatment options—he or she may keep you on your current therapy or recommend a treatment change

Your AR-V7 status can change

Some patients may develop resistance to AR-targeted therapies over time. Drug resistance means that the cancer has changed and the drugs no longer work like they are supposed to.

It's simple to determine your AR-V7 status



Get blood drawn



Receive results

Sent to your doctor in about 7 days



Discuss results



Decide next steps